

Allocation Chart „16 A-D Classical + 4 Bleach shades“
(staining technique, Zenostar Art Stains powder or paste)

A-D Classical	Zenostar Zr Translucent	Zenostar Art Stain	Zenostar Art Incisal Stain
A1	light (sun [®])	A1/A2/A3	*Grey-Violet / Transpa-Blue
A2	light (sun [®])	A1/A2/A3	
A3	medium (sun [®])	A1/A2/A3	
A3,5	intense (sun chroma [®])	A3,5	
A4	intense (sun chroma [®])	A4	
B1	light	B1/B2	
B2	light (sun [®])	B1/B2	
B3	medium (sun [®])	B3/B4	
B4	medium (sun chroma [®])	B3/B4	
C1	light	C1/C2/C3	
C2	medium	C1/C2/C3	
C3	medium	C1/C2/C3	
C4	intense	C4	
D2	medium (sun [®])	D2/D3	
D3	medium (sun [®])	D2/D3	
D4	medium	D4	
BL1	pure	BL1/BL2	
BL2	pure	BL1/BL2	
BL3	pure	BL3/BL4	
BL4	light	BL3/BL4	

[®] Zenostar Zr Translucent sun and sun chroma can be used alternatively for the reproduction of this tooth shade. The use of the new Zenostar Art Stains (Zenostar Art Module Focus) is required.

* The Zenostar Art Incisal Stains (Grey-Violet / Transpa-Blue) are used individually as required and depending on the tooth shade.



Allocation Chart "26 3D** shades" (staining technique, Zenostar Art Stains
3D-1 to 3D-7 only available in powder form)

Vita 3D shade**	Zenostar Zr Translucent	Zenostar Art Stain	Zenostar Art Incisal Stain
1M1	light	A1/A2/A3	*Grey-Violet/Transpa-Blue
1M2	light	A1/A2/A3	
2L1,5	light	B1/B2	
2L2,5	medium	B1/B2	
2M1	sun	3D-1	
2M2	sun	A3,5	
2M3	light	3D-2	
2R1,5	sun	3D-1	
2R2,5	sun	B3/B4	
3L1,5	sun	D4	
3L2,5	sun	B3/B4	
3M1	light	3D-1	
3M2	light	3D-2	
3M3	medium	3D-2	
3R1,5	sun	D2/D3	
3R2,5	sun chroma	A3,5	
4L1,5	sun	C1/C2/C3	
4L2,5	sun	A4	
4M1	medium	3D-3	
4M2	intense	3D-4	
4M3	sun	3D-5	
4R1,5	sun chroma	3D-6	
4R2,5	sun	3D-7	
5M1	sun chroma	3D-3	
5M2	sun chroma	3D-7	
5M3	sun chroma	3D-5	

The reproduction of the 16 A-D + 4 Bleach shades and the 26 3D shades** is summarized in the two tables above. The staining scheme is specifically adapted to the pre-shaded Zenostar Zr Translucent Blanks light, medium, intense, sun and sun chroma. The Zenostar Art 3D stains are available as single add-ons.

* The Zenostar Art Incisal Stains (Grey-Violet / Transpa-Blue) are used individually as required and depending on the tooth shade.

** VITA System 3D-Master is a registered trade mark of Vita Zahnfabrik H. Rauter GmbH & Co. KG, 79704 Bad Säckingen